Installation instructions

CardioMem[®] CM 100 XT @ CardioDay[®]

Revision 02

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1 Prerequisites

For using the CardioMem CM 100 XT with CardioDay the following prerequisites have to be met:

- CardioDay Version 2.5 SP3 and above
- the CardioDay **Event recorder** option
- **Windows 10** Operating System
- Bluetooth dongle LogiLink BT0015


Note: These installation instructions do not replace the operating manuals of CardioMem CM 100 XT and CardioDay. Always use the respective operating manuals to use both products.

Note: As long as the CardioMem CM 100 XT is in connection with CardioDay the recorder can not be used in the Tele ECG mode. The mode can be changed using the CM 100 Configurator software.


2 Bluetooth pairing of recorder and PC

Proceed as follows to pair the recorder with Windows:


1. Switch on the recorder.
2. Right-click the Windows Bluetooth-Icon.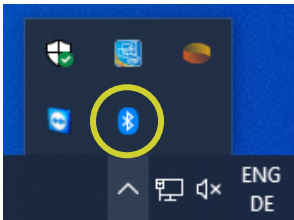


3. Right-click the **Add Bluetooth device** option.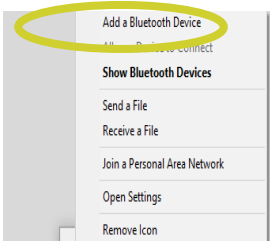


4. Enable Bluetooth on the recorder by pressing the key until a tone sequence sounds and the LED of the recorder flashes blue.
5. Click the „+“ button.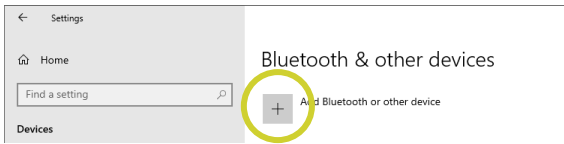


6. Select **Bluetooth**.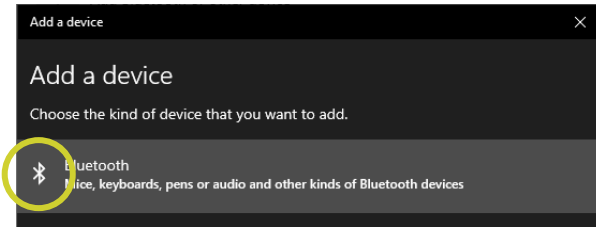

After a few seconds the recorder will be displayed with its serial number in the list.

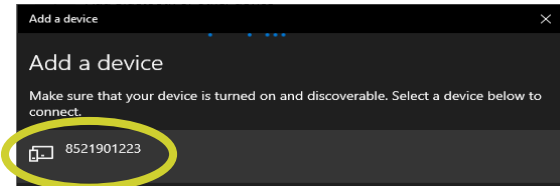
7. Click the entry of the recorder in the list, so that Windows 10 performs the pairing.
After successful pairing, a beep will sound on the recorder and the blue LED will flash fast.
8. **While the blue LED is still flashing**, press the button on the recorder to confirm the pairing.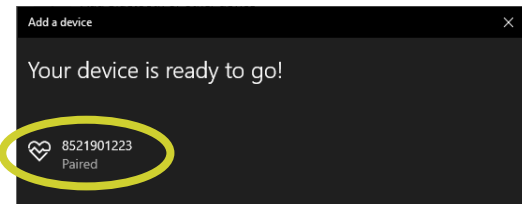

The recorder acknowledges your confirmation of the pairing with a tone sequence.

The recorder will now be displayed with its number under **Bluetooth & other devices**.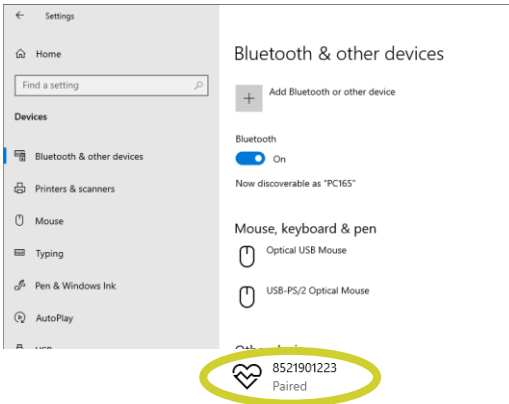


3 Preparing the recorder (Hookup)

Follow these steps to prepare the recorder in CardioDay for recording:

1. Start CardioDay.

For preparing the recorder also observe the instructions in the chapter "The Recorder menu" of the CardioDay Operating manual.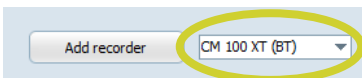


2. Select **CM 100 XT (BT)** to add a new recorder.
3. Click **Add recorder**.

A window opens displaying all recorders found in the vicinity. Be sure that Bluetooth is enabled on both the PC and the recorder or that the associated Bluetooth dongle is used. Follow all instructions in the operating manual of the recorder. It may take up to a minute to find all of the detectable recorders within range. Recorders that have been found will be added to the recorder selection list.

The **New** column in the **Find device** list indicates if a recorder has not yet been added to the recorder list.

- Select the desired recorder from the list and click the **Add** button.

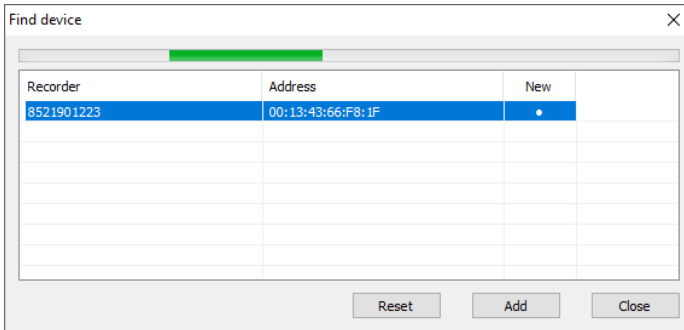The recorder is now included in the recorder list.

Patient ID	Last name	First name	Connect date	Serial number	Type	MAC
				8521901223	CM100	00:13:43:66:F8:1F

- Click the **Connect** button to open the **Hookup** window.

Click the **Transfer PC time** button to set the clock of the recorder to the current computer time.

The battery state is displayed in the Hookup main screen on the left-hand side of the electrode placement window. If the symbol is red, it indicates that the battery does not have enough capacity for the selected recording duration. The battery will need to be replaced.

- Click the **Options** button in the upper right of the window.
A window opens where you can set the recording parameters.

The screenshot shows a dialog box titled "Options" with a close button (X) in the top right corner. It contains several settings, each with a dropdown menu:

- Pre/post: 45 s / 15 s
- Brady: 40 bpm
- Tachy: 140 bpm
- Pause: 2.5 s
- AFib: On
- TIR: 2 h

At the bottom of the dialog box are two buttons: "OK" and "Cancel".

- Check the signal quality in the online ECG which is displayed next to the patient data.
The electrode placement guide is displayed on the right below the online ECG.
- If the signal quality is sufficient click the **Start recording** button.
- Click the **Close** button to close the online connection and the window.

4 Resetting the recorder


If there are patient records and events on the recorder that have already been saved and should be removed you can reset the recorder by following these steps:

- Open the **Hookup** window in CardioDay and enable Bluetooth on the recorder.
- Select **CM 100 XT (BT)** to reset a recorder.

The screenshot shows a section of the "Hookup" window. It features a button labeled "Add recorder" and a dropdown menu currently displaying "CM 100 XT (BT)".

- Click the **Add recorder** button.

4. Select the recorder in the list and click the **Reset** button.

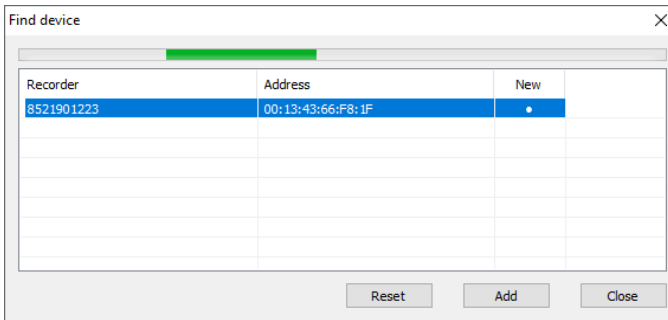A message is displayed indicating that the recorder has been successfully reset.

Note: The recorder options previously set remain and will not be reset.

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Medizin- und Informationstechnik AG